Aquatec® Step

en	Stepping aid User Manual	2
de	Einstiegshilfe Gebrauchsanweisung	4
fr	Marchepied Manuel d'utilisation	6
it	Ausilio per salita Manuale d'uso	8
es	Escalón de ayuda Manual del usuario	10
pt	Degrau anti-derrapante Manual de utilização	12
nl	Opstapje Gebruiksaanwijzing	14
sv	Pall Bruksanvisning	16
no	Badekrakk Bruksanvisning	18
da	Skammel Brugsanvisning	20
fi	Korokeapuväline Käyttöohje	21
hr	Pomagalo za zakoračivanje Upute za uporabu	23

This manual MUST be given to the user of the product.
BEFORE using this product, this manual MUST be read and saved for future reference.

AQUATEC®

Yes, you can.®

?.1 Images

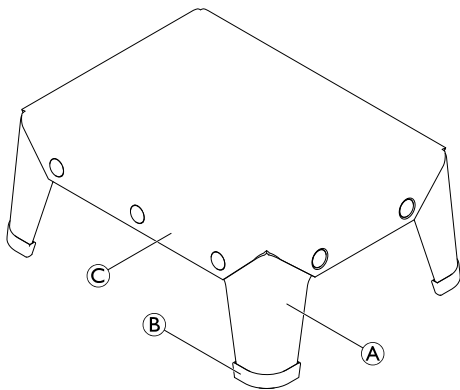
Fig. 1

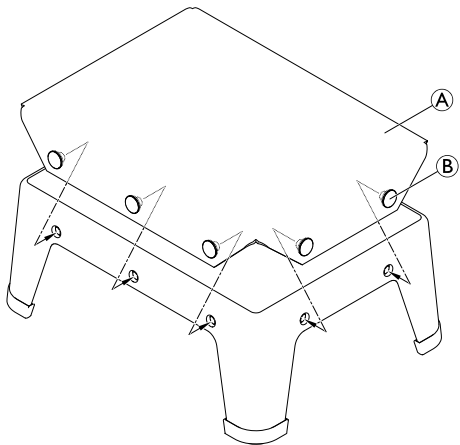
Fig. 3

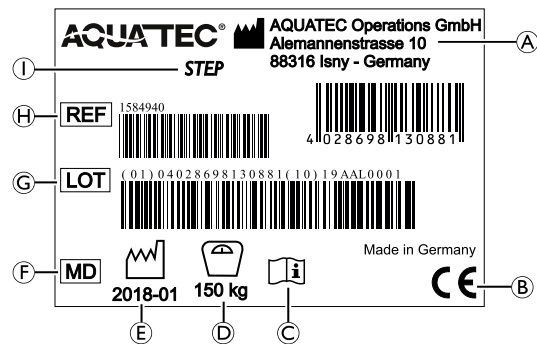
Fig. 2

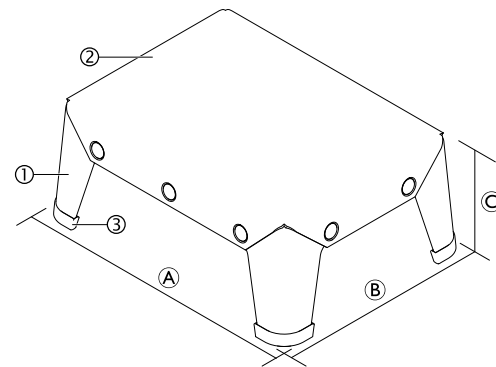
Fig. 4

1 Introduction

This user manual contains important information about the handling of the product. To ensure safety when using the product, read the user manual carefully and follow the safety instructions.

Invacare reserves the right to alter product specifications without further notice.

Before reading this document, make sure you have the latest version. You find the latest version as a PDF on the Invacare website.

If you find that the font size in the printed document is difficult to read, you can download the PDF version from the website. The PDF can then be scaled on screen to a font size that is more comfortable for you.

In case of a serious incident with the product, you should inform the manufacturer and the competent authority in your country.

2 Warranty Information

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries.

Warranty claims can only be made through the provider from whom the product was obtained.

3 Compliance

Quality is fundamental to the company's operation, working within the disciplines of ISO 13485.

en

This product features the CE mark, in compliance with the Medical Device Regulation 2017/745 Class I. The launch date of this product is stated in the CE declaration of conformity.

We are continuously working towards ensuring that the company's impact on the environment, locally and globally, is reduced to a minimum.

We only use REACH compliant materials and components.

3.1 Product-specific standards

The product has been tested and conforms to EN 12182 (Assistive products for persons with disability - general requirements and test methods) and related standards.

For further information about local standards and regulations, contact your local Invacare representative. See addresses at the end of this document.

4 Service life

The expected service life of this product is three years when used daily and in accordance with the safety instructions, maintenance intervals and correct use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

5 Limitation of Liability

Invacare accepts no liability for damage arising from:

- Non-compliance with the user manual
- Incorrect use
- Natural wear and tear
- Incorrect assembly or set-up by the purchaser or a third party
- Technical modifications
- Unauthorised modifications and/or use of unsuitable spare parts

6 Safety

6.1 Safety information

WARNING!

Risk of falling!

- Do not use if defective.
- Do not make any unauthorized alterations or modifications to the product.
- Do not move or displace the product under load or during use.
- Only use the product on level ground.
- Only use accessories intended for this product.

WARNING!

- Before first use, clean the product thoroughly.
- Before every use, check the product for damage and ensure that it is securely assembled.

This product does not require maintenance.

6.2 Labels and symbols on the product (Fig. 2)

Ⓐ	Manufacturer	Ⓕ	Medical Device
Ⓑ	European Conformity	Ⓖ	Lot number
Ⓒ	Read user manual	Ⓗ	Reference number
Ⓓ	Load capacity	Ⓘ	Product designation
Ⓔ	Production date		

The identification label is attached to the underside of the product.

7 Intended use

This stepping aid is intended as an aid to enter into or get out of a bathtub for persons with mobility limitations. The maximum user weight is 150 kg.

Indication: If used without an assistant, the user should be able to keep his balance. There are no contraindications known, when using the product as intended.

Operating temperature: 10 – 40 °C

8 Scope of delivery (Fig. 1)

The Aquatec® Step is available with or without the non-slip, removable cover option.

The following components are included in the scope of delivery:

Ⓐ	Frame
Ⓑ	Non-slip, removable rubber caps
Ⓒ	Non-slip, removable cover (optional)
	User manual

Ⓐ, Ⓑ and optionally Ⓒ are pre-assembled.

9 Removing and installing the non-slip cover (Fig. 3)

1. To remove unbutton the cover Ⓐ from the product.
2. To install rebutton the cover.

WARNING!

Risk of slipping

- Make sure that all 10 fastening buttons Ⓑ are securely fastened before using the product.

10 Cleaning and Disinfection

This product is maintenance-free, provided that the cleaning and disinfection instructions are adhered to.

10.1 General safety information

CAUTION!

Risk of contamination

- Take precautions for yourself and use appropriate protective equipment.

IMPORTANT!

Wrong fluids or methods can harm or damage the product.

- All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean.
- Never use corrosive fluids (alkalines, acid etc.) or abrasive cleaning agents. We recommend an ordinary household cleaning agent such as dishwashing liquid, if not specified otherwise in the cleaning instructions.
- Never use a solvent (cellulose thinner, acetone etc.) that changes the structure of the plastic or dissolves the attached labels.
- Always make sure that the product is completely dried before taking it into use again.

For cleaning and disinfection in clinical or long-term care environments, follow your in-house procedures.

10.2 Cleaning Intervals

IMPORTANT!

Regular cleaning and disinfection enhances smooth operation, increases the service life and prevents contamination.

Clean and disinfect the product

- regularly while it is in use,
- before and after any service procedure,
- when it has been in contact with any body fluids,
- before using it for a new user.

10.3 Cleaning

1. Remove the non-slip cover (if included) and machine-wash at max. 60 °C.
2. Pull off the non-slip rubber caps.
3. Clean the frame and the non-slip rubber caps by wiping down all generally accessible surfaces.
4. Dry the parts with a cloth.
5. Reinstall the rubber caps and the cover.

10.4 Disinfection instructions

Information on recommended disinfectants and methods can be found on <https://vah-online.de/en/for-users>.

In domestic care

Method: Follow the application notes for the used disinfectant and wipe disinfect all accessible surfaces.

Disinfectant: Ordinary household disinfectant.

Drying: Allow the product to air-dry.

In clinical and long term care

Follow your in-house disinfection procedures and only use the disinfectants and methods specified therein.

11 Re-use

The product is suitable for repeated use. The number of times it can be used depends on how often and in which way the product is used. Before reuse, clean and disinfect the product thoroughly

(→ *Cleaning and disinfection*).

12 Disposal

Be environmentally responsible and recycle this product through your recycling facility at its end of life.

Disassemble the product and its components, so the different materials can be separated and recycled individually.

The disposal and recycling of used products and packaging must comply with the laws and regulations for waste handling in each country. Contact your local waste management company for information.

13 Technical data (Fig. 4)

Dimensions and weight		Materials	
Ⓐ	410 mm	①	PVC
Ⓑ	310 mm	②	PP
Ⓒ	165 mm	③	TPE

Product weight: 1.5 kg

Max. user weight: 150 kg

All product components are corrosion resistant and do not contain natural rubber latex.

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Invacare Limited
Aquatec Operations GmbH